

# MAGNETIC ALLEN PENCIL

The Magnetic Allen Pencil is “charged” using an exclusive rare earth magnet. Once charged, it then uses magnetic forces to latch onto the screw. The screw is easily retrievable out of the kit and remains on the Magnetic Allen Pencil for placement and tightening within the patient’s mouth.


## Magnet Instructions:

- STEP 1:** Slide the charging magnet onto the Allen Pencil tip to activate the magnetic force.
- STEP 2:** After 2-3 seconds, remove the charging magnet and set aside.
- STEP 3:** Proceed with CS® System installation using magnetically charged Allen Pencil.

## Additional Notes:

- Charging magnet is cold sterilizable only. Allen Pencil is heat sterilizable only.
- Once Magnet and Allen Pencil are sterilized, magnet can be stored engaged with Allen Pencil.
- Important: Keep magnet away from electronics, damage may occur.

**Fast & Reduces Chairtime**

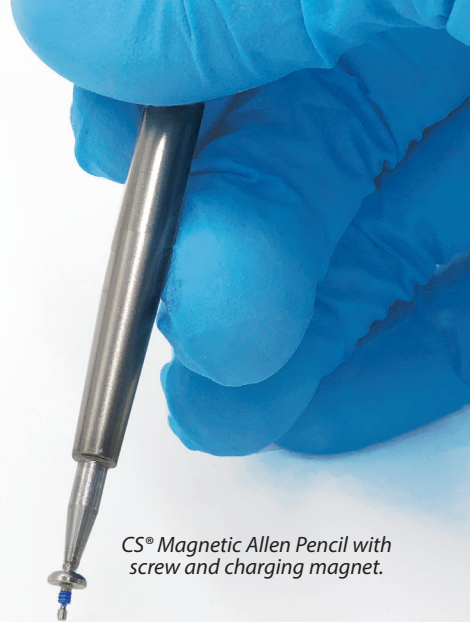

*CS® Magnetic Allen Pencil with screw and charging magnet.*

