



INDICATIONS FOR USE: Orthodontic Prophylaxis Pumice for use in cleaning the tooth surface prior to acid etching.

PRODUCT INFORMATION:

Pre-packaged and pre-mixed orthodontic prophylaxis pumice.

No flavoring, fluorides, oils or chemicals which could leave a film that may inhibit or weaken the bond.

1st & Final[®] prophylaxis pumice can be used prior to bonding to achieve a thorough prophylaxis; or after appliance removal to achieve a clean fresh surface.

INSTRUCTIONS FOR USE

1. Remove foil barrier from 1st & Final[®] prophylaxis cup.
2. Using a prophylaxis angle (Reliance Prep Step[™] Disposable Prophylaxis Angle), fill the prophylaxis angle cup with paste and establish a firm finger rest.
3. Bring the prophylaxis cup to close proximity to the tooth surface before operating the prophylaxis angle.
4. Replenish the supply of paste in the prophylaxis cup, as needed. Do not overfill.
5. After each patient, dispose of prophylaxis angle and unused prophylaxis cup of 1st & Final[®]. Do Not Re-use.



Single Use Only. Each individual unit of 1st & Final[®] contains sufficient paste for one full prophylaxis procedure.



Excessive prophylaxis paste should be expectorated, not swallowed. Will cause eye irritation. May cause skin irritation. Prolong exposure may cause respiratory system irritation. Ingestion of large amounts may cause nausea, vomiting, headaches, or dizziness. Refer to SDS for additional information.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.



Temperature Limitation: Store device at 15-30 Degrees C).

Rx Only: Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com

Made in the USA

Check out our new learning
platform Reliance U!
www.RelianceU.tech

EC

REP

EMERGO EUROPE

Prinsessegracht 20
2514 AP The Hague
The Netherlands



MD

Australian Sponsor:
Emergo Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia

Scan for more information