

Norris 20/26[®] CLEAR

BRACKET SYSTEM

With The Norris 20/26[®] Ceramic Bracket System, Orthodontists Finally
Have A Clear Self-Ligating Bracket That Performs Like Metal.



U/L 5X5 BRACKET SET
Item #220-01C5



CLEAR BRACKET OPENING TOOL
Item #220-5005



HIP TOOL
Item #216-5000



CLEAR DEBONDING PLIER
Item #216-5025



INFINITY AESTHETIC WIRE




INTRO KIT

UPPER & LOWER 5X5

#220-01C5-INT

Each Intro Kit Contains **10** 5x5 Bracket Sets
And The Following Complimentary Items:

- 1x** .014 Infinity Aesthetic NiTi Extra Broad (#10-216-200DW)
- 1x** .018 x .018 Infinity Aesthetic NiTi Extra Broad (#10-226-550DW)
- 1x** .019 x .025 Infinity Aesthetic NiTi Extra Broad (#10-426-700DW)
- 1x** Clear Bracket Opening Tool (#220-5005)
- 1x** HIP Tool - Wire Seater (#216-5000)
- 1x** Clear Debonding Plier (#216-5025)

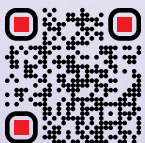

Scan To Discover
The Future Of
Ceramic Brackets.

DynaFlex[®]

Contact Ralph Esguerra, Vice President of International Sales
+1-760-300-9764 | ralphe@dynaflex.com | www.dynaflex.com


Norris 20/26 CLEAR

Norris Clear SL Initial User Protocols

Opening & Closing Brackets

A slimmer, single-ended **Clear Bracket Opening Tool (#220-5005)** works seamlessly to open Norris Clear SL doors. The current standard openers, are slightly too thick and **should not be used**. Please avoid using a scaler or pulling to open the door.

Please Note: Doors may feel slightly stiffer during closure due to the tight tolerances.

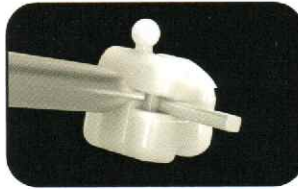

STEP 1:
Insert the **Clear Bracket Opening Tool (#220-5005)** horizontally into the door slot.



STEP 2:
Rotate the tool **90 degrees** in either direction (i.e. turning a key in a door).



STEP 3:
Ensure the wire is **fully seated** before closing **with your finger**. **Avoid Weingarts** when closing doors.

DynaFlex®

www.dynaflex.com | 866-346-5665

040626 © 2026 DynaFlex®, Lake St. Louis, MO 63367. Printed in U.S.A.

Archwire Sequencing Guide

PHASE	INITIAL ALIGNMENT	WORKING ALIGNMENT	WORKING FINISHING I	WORKING FINISHING II
WIRE	.014" NiTi	.018" x .018" NiTi	.019" x .025" Rounded NiTi	.019" x .025" Rounded Beta
DURATION	10 Weeks	10 Weeks	10 Weeks	8 Weeks Or Until Finished

Important Instructions For Norris Clear Brackets

Allow the **.014" NiTi** wire to fully express before progressing.

The **maximum** recommended **stainless steel wire is .017" x .025"**.


Ensure the **.018" x .018" NiTi** wire has resolved all rotations before transitioning to **.019" x .025" NiTi**.

Beta wire should only be used **after one month** of .019" x .025" NiTi wear.

Repositioning & Final Debonding

Similar to the Norris Metal brackets, the Norris Clear Brackets have a notably high bond strength. Please use the **Clear Debonding Plier (#216-5025)** for debonding (detailed instructions will be provided). Please remember to close all of the bracket doors before debonding. Ensure all flash is properly removed. Repositions can be performed just like the metal brackets. We recommend that any repositioning is done while the patient is still in NiTi wire for best results.

Position Plier Occlusal-Gingival

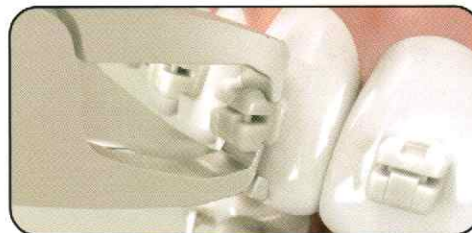

DynaFlex®

www.dynaflex.com | 866-346-5665

040626 © 2026 DynaFlex®, Lake St. Louis, MO 63367. Printed in U.S.A.