

## Quick and Easy


## STEP-BY-STEP INSTRUCTIONS

### TRADITIONAL PIVOT SETUP

#### **CLASS II**

# **UPPER ARCH**

#### **CLASS III**

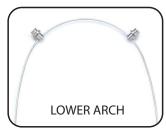

#### STEP 1:

Place 2 pivots on the lower archwire, position pivots distal to the lower canines, insert and ligate archwire

#### STEP 2:

STEP 1:

Outside the mouth, assemble a spring, screw and pivot combination. Place assembled components onto the *lower* archwire. Insert and ligate archwire positioning the pivots mesial to the *lower* first molars

Place 2 pivots on the upper

distal to the upper canines,

insert and ligate archwire

archwire, position pivots





#### STEP 2:

Outside the mouth, assemble a spring, screw and pivot combination. Place assembled components onto the upper archwire. Insert and ligate archwire positioning the pivots mesial to the upper first molars



Using a hemostat, stretch spring eyelet onto the upper pivot and secure in place with a screw





#### **STEP 3:**

Using a hemostat, stretch spring eyelet onto the lower pivot and secure in place with a screw









