## **TWIST-LOCK** PIVOT

## Helpful Application Tip







## **DynaFlex**°

www.dynaflex.com | 866-346-5665

## Follow the steps in order

- Having trouble with your CS®5 Twist-Lock Pivot rolling to one side or the other on those smaller gaged wires?
- Place a small piece of orthodontic brace wax on the occlusal/incisal edge for the upper and gingival side for the lower when working on a patient. This wax can help to hold the CS<sup>®</sup>5 Twist-Lock Pivot in place while you assemble the remaining parts.
- The wax will hold the CS\*5 Twist-Lock Pivot in place while you use the Magnetic Allen Pencil to tighten the screw that holds the CS\* spring in place.
- Once you have completed your CS®5 assembly, you can remove the orthodontic brace wax with a scaler.