  
RELIANCE ORTHODONTIC PRODUCTS, INC.  
1540 W. Thorndale Avenue,  
Itasca, IL 60143 USA  
1-800-323-4348 | Made in USA | Rx Only  
RelianceOrthodontics.com

 RelianceOrthodontics.com

 ITC TRANSLATIONS  
ITC USA  
2000 PGA Blvd., Suite 4440  
Palm Beach Gardens, FL 33408

 EC REP MD

**EMERGO EUROPE**  
Westervoortseindj 60  
6827 AT Arnhem  
The Netherlands

**AUSTRALIAN SPONSOR**  
**EMERGO AUSTRALIA**  
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

**U.K. PERSON RESPONSIBLE**  
**EMERGO CONSULTING (UK) LIMITED**  
c/o Cr360 – UL International  
Compass House, Vision Park Histon  
Cambridge  
CB24 9BZ  
England, United Kingdom

**SWITZERLAND AUTHORIZED REPRESENTATIVE**  
MedEnvoy Global BV  
Leidschendam-Voorburg, Zug Branch Office  
Gotthardstrasse 28, 6302 Zug, Switzerland

**RELIAMOLD®**  
**MINI MOLDS**

## INSTRUCTIONS FOR USE

 V2 OCT.2023  
1639

 **RELIANCE®**  
ORTHODONTIC  
PRODUCTS

SCAN FOR MORE INFORMATION

### EN - ENGLISH

**INDICATIONS FOR USE:** ReliaMolds are malleable, single-use molds designed to adapt to tooth structure to form various types of composite for functional Orthodontic treatment.

**PERFORMANCE INDICATOR:** Accurately forms composite as described.

### INSTRUCTIONS FOR USE

**PROPHYLAXIS:** Prophy the enamel surface. Rinse and dry thoroughly.

**ETCH:** Etch the enamel surface, rinse and dry.

**APPLICATION:** Apply a thin coat of Assure Plus and lightly dry (tack light curing at this stage can help reduce mold drift). With an instrument, hold the male end of molds. Fill mold cavity with desired light cure material (FlowTain, LCR, GoTo) and place on tooth surface gently. (Note: if the intention is to bond to a thermoformed aligner, Bond Aligner must be used) Cure for the appropriate duration according to specific light intensity.

**PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately.

Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

**⚠ Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.**

**② Do not reuse.**

**⚠ Keep away from sunlight.**

**PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

**CONTRAINDICATIONS:** Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

**SAFE DISPOSAL:** Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous-waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

**TO REORDER:** #RMK, #RMBR3, #RMBR5, #RBMU, #RMC, #RMH, #RMMP, #RMWB, #RMWBS, #RMT, #BHA

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