

PhotoMed/Canon SX730/SX740 Closeup Attachment Kit - Installation

These instructions will show you how to install the PhotoMed Closeup Attachment Kit on your Canon SX730/SX740 camera. If you need any help, please call PhotoMed at 800-998-7765.




The Closeup Attachment Kit includes:

- 1 - camera mounting fork
- 1 - mounting tube, diffuser and macro lens (*as one piece*)
- 1 - screwdriver (*not shown*)
- 1 - PhotoMed SX730/SX740 Quick Start Guide (*not shown*)



The LCD screen on the back of the SX730/SX740 camera is hinged.

From the bottom of the camera, tilt the screen up slightly (*as shown at left*).

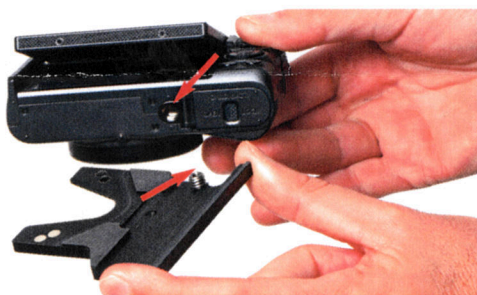

The Mounting Fork attaches to the bottom of the camera via the tripod socket. The screw in the Mounting Fork threads into the tripod socket.



Position the Mounting Fork as shown in the photo. There is a raised lip (*indicated by the red arrow in the photo*) that should be aligned with the bottom edge of the camera.



Use the included Phillips head screwdriver to tighten the screw into the camera's tripod socket. Tighten fully until the Mounting Fork is securely attached to the camera.

The PhotoMed SX730/SX740 Quick Start Guide walks you through all of the necessary camera settings. If you have any questions, please contact us at 800-998-7765.

PhotoMed

14141 Covello St. #7C, Van Nuys, CA 91405
800-998-7765 ° 818-908-5369 ° FAX 818-908-5370
www.photomed.net ° info@photomed.net

© 2017, PhotoMed International. All rights reserved