

## Reliance

## **GEL ETCHANT IN SYRINGE**

<u>INDICATIONS FOR USE:</u> Reliance Gel Etchant is 37% phosphoric acid (H3PO4) polymer-thickened and intended for use on enamel to prepare the surface for bonding.

## **INSTRUCTIONS FOR USE**

- 1. Unscrew and remove the white storage cap from the tip of the syringe.
- Snap the needle tip dispenser in place and twist clockwise until needle is locked.
- Isolate the teeth for etching. Apply etching agent onto the entire area to be bonded. Allow 30 seconds for etching.

CAUTION: Avoid placing etching agent on soft tissue.

- 4. Rinse thoroughly and air dry
- 5. When procedure is completed, remove needle tip, discard needle tip and replace the white storage cap.

CAUTION: Etching gels (contain phosphoric acid) are severe eye and tissue irritants. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Injury may result if etchant is allowed to remain on the skin or mucosa for extended periods of time.

CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPS BEFORE EACH PATIENT. TIPS ARE SINGLE USE DEVICES. A NEW TIP IS REQUIRED FOR EACH PATIENT.

TEMPERATURE LIMITATION: Store at room temperature (15-30 °C)
TO REORDER: REF# EGSS- 6.5q Gel Etchant in Syringes w/Tips,

REFf#E22TIPS50-50 22 Gauge Tips (Tips only)

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**Rx Only:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A. 1-800-323-4348 · www.RelianceOrthodontics.com



Made in the USA

Scan for more information

Check out our new learning platform Reliance U! www.RelianceU.tech



EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands



Australian Sponsor: Emergo Australia 201 Sussex St. Darling Park Tower II, Level 20, Sydney, NSW 2000 Australia